EN

F-315A
Instruction Manual
WE-2018-F315A

ROTARY BRUSH

WARNING

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.

- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.

- Never use the unit outdoors; exposure to the elements can cause damage.

- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

EC-DECLARATION OF COMPLIANCE

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU

2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU

RECYCLING

 Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to

incorrect operation.






The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

PACKAGE CONTENTS


- These components are subject to change without prior notice.

- Appearance of product may differ from the image.

No.	PRODUCT	REF.	QTY.
1	BRUSH HANDLE	10074-3	1
2	BRUSH HEAD 1	10073	1
3	BRUSH HEAD 2		1
4	BRUSH HEAD 3		1
5	BRUSH HEAD 4		1
7	MAIN UNIT	-	1
8	FEEDER CABLE	-	1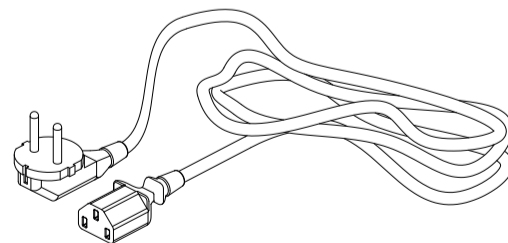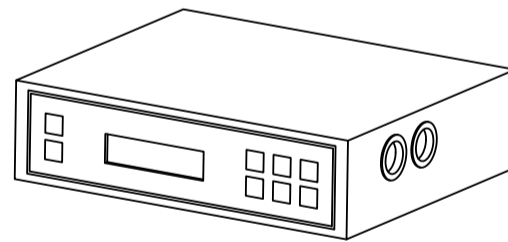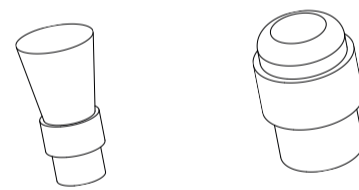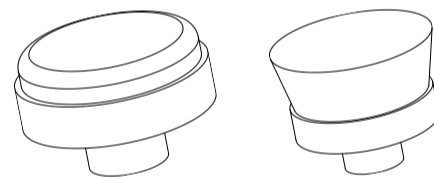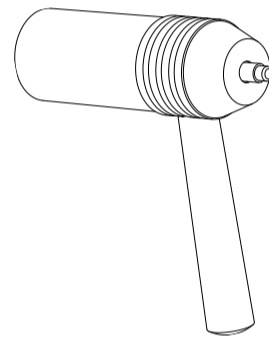

FRONT PANEL

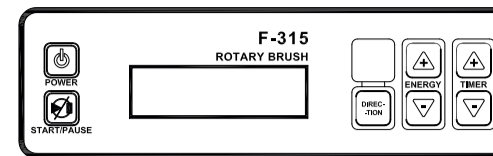INSTRUCTIONS OF USE

BEFORE TREATMENT:

1. Scrub the skin surface with a wet pad.

2. Using a steamer (or hot cloth) soften the skin surface to be treated.


3. After skin-cleansing maintenance, it is advisable to daub some nutritional oil on the target skin surface.

STARTING TREATMENT:

1. Connect the power supply via the rear panel.

2. Connect the accessories properly in the lateral panel.

3. Choose the appropriate brush head according to the customer's skin type. In general, delicate or dry skin needs a soft brush head, while harder brushes can be used for rough or greasy skin.

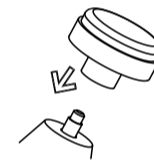4. Turn on the "POWER" button.

5. Adjust the desired "ROTATION", "ENERGY" and "TIMER".

6. Press the "START/PAUSE" button to start the treatment.

7. While holding the brush handgrip, adjust "ENERGY". The speed of rotation will increase as the adjusting angle is increased.

CAUTIONS

- Do not press the brush head excessively while it is running.

- Do not operate this instrument with any evident acid or alkaline substance.

- After the operation, turn the instrument off and disconnect the power cord.

- This function should not be used on customers who suffer from hypersensitive skin or dermatitis.

- If the capillary vessels in the skin are injured, do not cleanse the skin with this function

MAINTENANCE

- After the operation, submerge the brush head in clear water with a little cleanser for 5 minutes. Then, after rinsing it, submerge it in alcohol for half an hour.

- The brush head must be disinfected using ultraviolet radiation and then kept in its original packaging.

ACCESSORIES

REF.	DESCRIPTION
10074-3	BRUSH HANDLE
10073	BRUSH HEAD 1
	BRUSH HEAD 2
	BRUSH HEAD 3
	BRUSH HEAD 4
	BRUSH HEAD 5

TECHNICAL SHEET

Weights and sizes	Packing size (cm)	37x27,5x14
	Gross weight (kg)	3,5
	Product size (cm)	20,7x22x7
Working characteristics	Speed of rotation	0 - 550 RPM
	Preset working time	3 min
Electrical characteristics	Rated voltage	100V ~ 240V
	Frecuency	50Hz/60Hz
	Rated power	25 W

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