Manual de instrucciones

B-Equipment

F314B - New Series

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VERSION 1.24

Weelkome!

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Advertencia: Do not put the metal part of the tube in contact with water.

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Our products are subject to
continuous technical improvements.
Weelko reserves the right to
change or modify this manual and
its contents without prior notice.

Instructions Manual

B-Equipment

F314B - New Series

High frequency

VERSION 1.24

Weelkome!

Thank you for having acquired a WEELKO product and for the trust shown in us.

This beauty instrument has been designed to fulfill the most demanding work requirements of those professionals working in the beauty and aesthetics field, for face beauty treatments. This instrument should only be used by trained and skilled beauticians. This device should never be used for any purpose than its intended design limitations.

The company reserves the right to modify this manual with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail to info@weelko.com informing us so that we can correct them.

Use of the manual

Before using this beauty instrument, please read the instruction manual carefully. It is important to keep this manual and store it in a safe place so that it can be consulted at any time.

If you follow these instructions carefully and consider the safety measures detailed, you will avoid any problems when using the device.

Some texts and/or illustrations may not belong to your instrument, since they are general, and they may depend on your product choice at the purchase time.

Our company reserves the right to improve the design of the product and modify them without prior notice.

Warranty

The warranty period is effective from the date of purchase of the product. It covers all the components of the beauty instrument if the product is used under normal conditions and correct use. Please contact your point of sale to get further information on warranty terms and conditions. The warranty does not cover defects or malfunction caused by:

- Misuse, abuse or improper maintenance of the instrument.
- Non-observance of the precautions and warnings stated in this instruction manual.
- Lack of maintenance and maintenance or repair tasks carried out by non-qualified service personnel who are unauthorized by the manufacturer.
- Problems or breakdowns that may appear after non-authorized repairs performed by third parties, especially non-authorized personnel.
- Wrong and faulty electrical installations or which do not fulfil the legal requirements.
- The company is not liable for incidental or consequential damage to property or business.

EC Declaration of conformity

This product is produced and marked in conformity with the directives hereby stated:

Low voltage Directive: 2014/35/CE
Electromagnetic compatibility : 2014/30/CE

Waste management

The symbol of the crossed-out container means that it is legally required to dispose of the product separately from unsorted urban waste. It means that it must be disposed of separately from household waste.

The proper disposal of these products contributes to saving valuable resources and prevents negative effects on human health and the environment if not discarded properly.

For their proper treatment, recovery, and recycling, take these products to designated collection points or consult the local laws of your country.

Safety instructions

- This product is intended only for qualified and certified beauty practitioners.
- Do not use or store the product in a damp place such as bathroom to avoid malfunction and danger.
- Do not use this product near flammable items (e.g., alcohol, nail polish remover, sprays, etc.)
- Do not use this product under the following conditions: when the power cord or plug is damaged or when it gets hot, and when the plug in the power socket comes loose.
- Do not plug and unplug power cord connectors or use this product with wet hands.
- Please make sure that the working voltage of this product matches its rated voltage.
- Before its use, make sure that the power plug and the power cord connector are fully plugged in to avoid any danger.
- If the device does not have to be used, please turn it off and disconnect it from the power supply.
- Please regularly clean the dust accumulated on the power plug and power cord to avoid failure of insulation caused by moisture, dust, and any other factors.
- Be sure to use the special power cord assembly for this product. Do not use the power cord assembly on any other appliance.
- To keep the power cord in best condition, do not wrap it around the product.
- Keep this product out of reach of children. Do not allow them to use this product.
- Please do not modify, disassemble, or repair this product by yourself. When a repair is needed, please contact the manufacturer, agent, or professional maintenance personnel to check and repair it.
- If the plug is damaged and it must be repaired or replaced, please contact the distributor, the service center or related personal with appropriate skills and experience.
- Please be sure to use this product according to the purpose specified in this manual. Do not use accessories which are not recommended by the manufacturer.

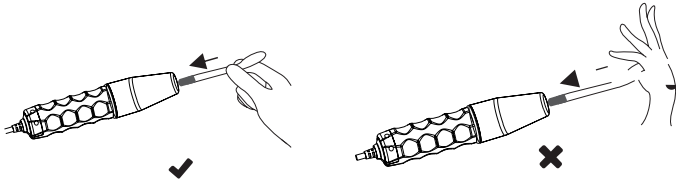
Safety during the treatment

- It is forbidden to touch the output interface of the main machine of the high-frequency electrotherapy beauty instrument or the output interface of the handle of the high-frequency electrotherapy when the main machine is powered on.
- Do not point the tube directly or in contact with the eyes a rea.
- This product is prohibited far the following people and parts:
 - *Pregnant women and those equipped with adjusters (e.g. cardiac adjusters).*
 - *People with vascular diseases and high skin sensitivity.*
 - *Having metal implants or metal dentures embedded in the body.*
 - *People with skin wounds and other conditions.*
 - *Body parts with thick hair.*
- Do not use skin lotions containing alcohol, as alcohol is highly flammable. If alcohol-based teneer is required, only apply it after using this product.
- Do not grasp the junction between the high-frequency handle and the electrode tube by hand during operation to avoid being shocked.
- The skin retention time of each part should not exceed 5 seconds.
- This function shall not be used far other than cosmetic purposes .
- Before use, check whether the electrode tube or the metal head on the tube is loose and whether the electrode tube cracks or is damaged. If so, do not use it and replace the electrode tube immediately.
- Reduce the current intensity to zero gradually, turn off the power switch, and pull out the power cord plug before inserting or pulling out the electrotherapy tube.
- When inserting or pulling out the electrode tube, first wear rubber gloves and hold the electrode tube with thumb and farefinger: insert the tube to the bottom of the hale until it is firm, and do not push the tube with the palm to prevent the glass from breaking and causing injuries; It is farbidden to pull out vigorously, so as to prevent the metal head on the electrode tube from falling off or breaking, which may cause injury.

Safety instructions

· You **MUST** wear rubber gloves and hold with your thumb and forefinger, then insert the electrode pipe till its tightly fitted.

· You **MUST** wear rubber gloves. It is strictly prohibited to push the electrode pipe with your palm.

· When in use, the high voltage current output from the electrotherapy tube will cause interference.

· It is forbidden to approach the host machine and other electrical appliances of the high-frequency electrotherapy beauty instrument. If it interferes with the host, it may cause the host to not work or crack.

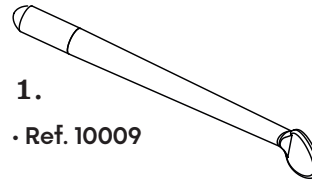
· When the electrode tube is less than 2.5cm from the skin but not in contact with the skin, the current intensity should be reduced immediately if sparks emits to irritate the skin. If the customer still feels the spark after the above operation, a small piece of dry cotton towel (nylon or other synthetic fiber towels are not allowed) put on the customer's face.

· When using, always keep the electrode tube on the customer's skin. High-frequency indirect care: when turning on and off the instrument, one hand must always be in contact with the customer's skin, never leave the customer's skin. Both sides feel the current and it causes discomfort.

· Turn off the power before pull out the power cord plug after use, and pull out the electrotherapy tube from the high-frequency electrotherapy handle, then sterilize and keep it dry before putting it into the accessory box.

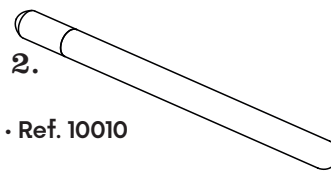
HF Electrodes

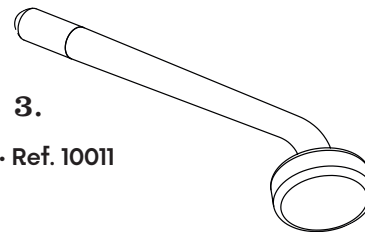
1.
· Ref. 10009

· Ladle-shaped electrode pipe: used for medium areas of skin (chin, etc). fig.1

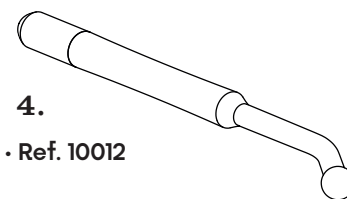
2.
· Ref. 10010

· Straight electrode pipe: used for indirect treatment.

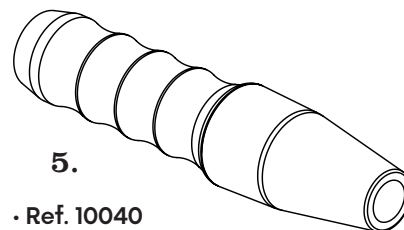
3.
· Ref. 10011

· Mushroom-shaped electrode pipe: used for large areas of skin (face, forehead, neck, etc).

4.
· Ref. 10012

· Ball-like electrode pipe: used for small areas of skin (nose, etc).

5.
· Ref. 10040

· High frequency handle. Electrode holder.

Product description

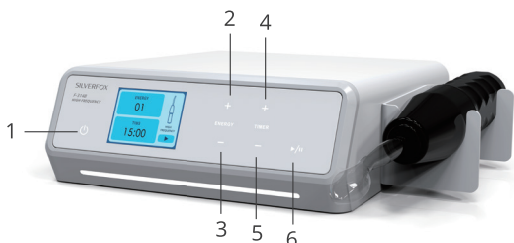
High Frequency instrument The package contains: main unit + 1 shelf + 1 + 1 power cord and the following probes:

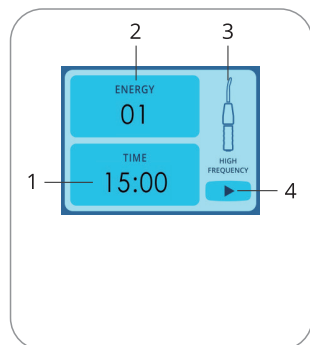
- High frequency handle
- Spoon shaped electrode
- Dot shaped electrode
- Rectangular electrode
- Mushroom shaped electrode

Screen & functions

1. Power switch
2. Power Energy (Increase +)
3. Power Energy (Decrease -)
4. Time increase +
5. Time increase
6. Start/Pause

1. Time
2. Energy level
3. Working status
4. Operating status

Operation

1. Insert the High-Frequency into the corresponding socket of the Equipment. When the equipment is connected to the power cable properly, the "Power switch" is displayed in red. Press the "Power Switch" to enter into the working mode and it would show in white in the screen. The equipment defaults to the continuous wave output state; thus the energy output of level 1 is set to 15 minutes.

2. There are various modes that can be selected based on your needs, choose the appropriate electrotherapy tube to install on the handle of high frequency electrotherapy.

3. Press the "Time increase/decrease button" which allows you to set the Working time from 1-30 minutes.

4. Press the "Start"/"Pause" button to start the operation. The power energy level is adjustable which allows being set from level 1 to 10.

5. When you want the Ultrasonic probe to stop working during the operation you can press the "Pause" button.

6. Press the "power switch" to turn off the equipment when the treatment is end, and the "power switch" will display red at this time. Clean the electrotherapy tube with 75% alcohol after the unplugging the power plug, and store in a dry place.

Recommended Using time

- 1) Oily skin: 10 to 15 minutes
- 2) Dry and dehydrated skin: 3 to 5 minutes

Warning: Do not put the metal part of the tube in contact with water.

Cosmetic purpose

High frequency instrument used to increase blood circulation, stimulate glandular activity, remove bacteria and ease the penetration of cosmetic products.

Technical sheet

B-Equipment reference	F-314B
Packaging dimensions (cm)	37 x 27,5 x 14 cm
Fonction	High frequency
Power	220-240 V
Frequency	50/60Hz
Rated power / Wattage	15W

Assistance

Only a technician authorised by Weelko is allowed to perform any kind of repairs on the bed. Please contact your distributor or point of sales for further information on technical assistance.

Troubleshooting

This troubleshooting guide is general for all the instruments belonging to F31X new series. Some of these faults and solutions may not apply to your device.

Fault	Reason	Solution
The main machine does not work	1. No power	1. Check whether the fuse in the power base is burned out. If burnt, replace the fuse. 2. Press the start button, the button will light up after it is powered.
	2. The power cord is not properly inserted or connected	2. Plug or connect the power cord properly.
The display screen does not work	1. No power	1. Turn on the power button (ON). 2. Contact your supplier.
The accessory probe does not work	1. Poor interface contact	1. Unplug the output cable, and then reinsert the socket.
Weak power for Cold & Hot hammer, scrubber, RF, Galvanic and other functions	1. The connector is not properly plugged in. 2. The accessories are too hot. Overheating can cause weak output power.	1. Adjust power to a higher level. 2. Wait for the accessories to cool down. Turn off the device when not in use.
Dermabrasion has weak suction	1. The thread of the diamond tip is not properly tightened. 2. The inside of the dermabrasion handle is obstructed. 3. The air tube is worn out. 4. The connection between the air tube and the suction handle is loose.	1. Adjust the power and the air regulation valve. 2. Replace the microdermabrasion accessories. 3. Replace the air hose with a new one, adjust the connecting piece, and change the filters.

Troubleshooting

Fault	Reason	Solution
Sprayer with weak spraying effect	<ol style="list-style-type: none"> 1. The joining screw of the airbrush with the spray bottle is not properly tightened. 2. The water receptacle is blocked. 3. The air tube is worn out. 4. The connection between the air tube and the suction handle is loose. 5. There is not enough water in the tank. 6. The hose is bent. 7. The vacuum is not connected. 	<ol style="list-style-type: none"> 2. Adjust the spray level to a higher one. 3. Replace the water receptacle. 4. Replace the air hose with a new one and adjust the connecting piece. 5. Check the water level in the bottle and add more when necessary. 6. Connect both handles.
The suction of the pen is weak	<ol style="list-style-type: none"> 1. The glass nozzle is not properly placed. 2. The suction handle is obstructed. 3. The air hose is worn out, damaged, or bent. 4. The connecting piece between the air hose and the handle has loosened. 	<ol style="list-style-type: none"> 1. Adjust the spray level to a higher one. 2. Replace the suction handle. 3. Replace the air hose with a new one and adjust the connecting piece.
The output of high frequency is weak	<ol style="list-style-type: none"> 1. The electrode is not properly placed. 2. Wear of the electrode rod due to excessive use. 	<ol style="list-style-type: none"> 1. Adjust the power to a higher level. 2. Replace the high-frequency accessories.

Pol. Ind. La Vallan
Camí de Garreta, 92
08600 Berga
Barcelona - Spain
T. +34 93 824 90 20
E. info@weelko.com

Our products are subject to
continuous technical improvements.
Weelko reserves the right to
change or modify this manual and
its contents without prior notice.