# Manual de instrucciones

## B-Equipment

F316B - New Series

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VERSION 1.24

## Weelkome!

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## Resolución de problemas

This troubleshooting guide is general for all the instruments belonging to F31X new series. Some of these faults and solutions may not apply to your device.

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---

## Weelkome!

Thank you for having acquired a WEELKO product and for the trust shown in us.

This beauty instrument has been designed to fulfill the most demanding work requirements of those professionals working in the beauty and aesthetics field, for face beauty treatments. This instrument should only be used by trained and skilled beauticians. This device should never be used for any purpose than its intended design limitations.

The company reserves the right to modify this manual with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail to [info@weelko.com](mailto:info@weelko.com) informing us so that we can correct them.

---

## Use of the manual

Before using this beauty instrument, please read the instruction manual carefully. It is important to keep this manual and store it in a safe place so that it can be consulted at any time.

If you follow these instructions carefully and consider the safety measures detailed, you will avoid any problems when using the device.

Some texts and/or illustrations may not belong to your instrument, since they are general, and they may depend on your product choice at the purchase time.

Our company reserves the right to improve the design of the product and modify them without prior notice.

---

## Warranty

The warranty period is effective from the date of purchase of the product. It covers all the components of the beauty instrument if the product is used under normal conditions and correct use. Please contact your point of sale to get further information on warranty terms and conditions. The warranty does not cover defects or malfunction caused by:

- Misuse, abuse or improper maintenance of the instrument.
- Non-observance of the precautions and warnings stated in this instruction manual.
- Lack of maintenance and maintenance or repair tasks carried out by non-qualified service personnel who are unauthorized by the manufacturer.
- Problems or breakdowns that may appear after non-authorized repairs performed by third parties, especially non-authorized personnel.
- Wrong and faulty electrical installations or which do not fulfil the legal requirements.
- The company is not liable for incidental or consequential damage to property or business.

---

## EC Declaration of conformity

This product is produced and marked in conformity with the directives hereby stated:

Low voltage Directive: 2014/35/CE  
Electromagnetic compatibility : 2014/30/CE

---

## Waste management

The symbol of the crossed-out container means that it is legally required to dispose of the product separately from unsorted urban waste. It means that it must be disposed of separately from household waste.

The proper disposal of these products contributes to saving valuable resources and prevents negative effects on human health and the environment if not discarded properly.

For their proper treatment, recovery, and recycling, take these products to designated collection points or consult the local laws of your country.

## Safety instructions

- This product is intended only for qualified and certified beauty practitioners.
- Do not use or store the product in a damp place such as bathroom to avoid malfunction and danger.
- Do not use this product near flammable items (e.g., alcohol, nail polish remover, sprays, etc.)
- Do not use this product under the following conditions: when the power cord or plug is damaged or when it gets hot, and when the plug in the power socket comes loose.
- Do not plug and unplug power cord connectors or use this product with wet hands.
- Please make sure that the working voltage of this product matches its rated voltage.
- Before its use, make sure that the power plug and the power cord connector are fully plugged in to avoid any danger.
- If the device does not have to be used, please turn it off and disconnect it from the power supply.
- Please regularly clean the dust accumulated on the power plug and power cord to avoid failure of insulation caused by moisture, dust, and any other factors.
- Be sure to use the special power cord assembly for this product. Do not use the power cord assembly on any other appliance.
- To keep the power cord in best condition, do not wrap it around the product.
- Keep this product out of reach of children. Do not allow them to use this product.
- Please do not modify, disassemble, or repair this product by yourself. When a repair is needed, please contact the manufacturer, agent, or professional maintenance personnel to check and repair it.
- If the plug is damaged and it must be repaired or replaced, please contact the distributor, the service center or related personal with appropriate skills and experience.
- Please be sure to use this product according to the purpose specified in this manual. Do not use accessories which are not recommended by the manufacturer.

## Safety during the treatment

- Do not operate suction function on eyes and surrounding skin.
- The following people are prohibited to use suction and spray functions:
  - Allergic patients;
  - Skin burns, scalds, sunburn, inflammation or other wounds.
- If there is no suction when the machine is started, it is a normal phenomenon. Wait for a moment and it can be restored to normal.
- If the spraying effect is not correct, you can adjust the pressure regulator at the end of the spray gun. Turn counterclockwise to increase pressure and clockwise to decrease it.
- Before each use, check whether the glass straw is cracked or broken. In such case, it should be prohibited to use and replace the glass straw immediately
- Glass straw can not be used in the same part for too long. Move suction pen fast, so as not to cause skin healing.
- Do not misconnect the nozzle into the suction jack, in case the skin care products are sucked into the air pump and the main machine is damaged.
- Do not pull the airway vigorously to avoid damage.
- After use, the power must be cut off.

# Product content


The packaging contains: main unit + 2 stands + 1 power cable + the following accessories:


- Suction handle
- Flat glass nozzle
- Round glass nozzle
- Pointed glass nozzle
- Spray gun with glass bottle.

# Screen & functions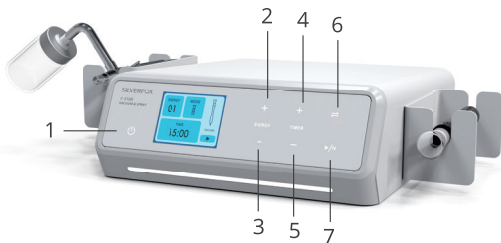

1. Power switch
2. Power Energy (Increase +)
3. Power Energy (Decrease -)
4. Time increase +
5. Time increase
6. Mode switch
7. Start/Pause

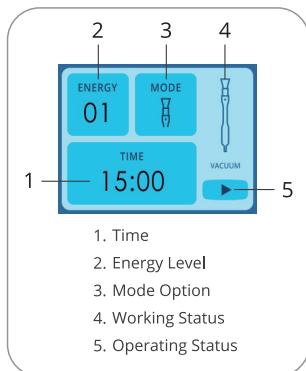1. Time
2. Energy level
3. Mode Operation
4. Working status
5. Operating status

1. Time
2. Energy Level
3. Mode Option
4. Working Status
5. Operating Status

# Operation

1. Insert the air hoses into the corresponding rear sockets of the main unit and connect them to each accessory (vacuum and spray). When the equipment is correctly connected to the power supply, the "Power switch" will display in red. Press the "Power switch" to enter the working mode, and the display will show in white.

2. The power is set by default. Level 1 is set to 15 minutes.

3. Press the mode change button to select the desired mode according to your needs; Suction or Hydration. If you choose the Suction mode, you should install the appropriate glass nozzle.

4. Press the "Increase/Decrease time" button, which allows you to set the working time from 1 to 30 minutes.

5. Press the "Start/Pause" button to start operation. The power level is adjustable and can be set from 1 to 10.

6. If you want to pause the equipment during treatment, press "Pause". When treatment is finished, press the "Power On/Off switch". At this point, the "Power On/Off switch" will display in red. Disconnect the device from the power outlet and clean the water replenishment instrument, suction handle, and glass nozzles with 75% alcohol. Store the accessories in a dry place.

## Recommended Using time

Do not exceed 10 minutes for suction. Hydrate properly.

# Cosmetic purpose

Deeply cleanses and exfoliates the skin. Improves blood circulation and tones the skin. Facilitates the absorption of cosmetic products. Reduces acne and eliminates blackheads.

## Technical sheet

B-Equipment Reference	F-316B
Product dimensions	20.8 x 23 x 7.5 cm
Packaging dimensions (cm)	37 x 27,5 x 14 cm
Function	Vacuum & Spray
Power	220-240 V
Frequency	50/60Hz
Power/Wattage	65W

## Assistance

Only a technician authorised by Weelko is allowed to perform any kind of repairs on the bed. Please contact your distributor or point of sales for further information on technical assistance.

## Troubleshooting

This troubleshooting guide is general for all the instruments belonging to F31XB new series. Some of these faults and solutions may not apply to your device.

Fault	Reason	Solution
The main machine does not work	1. No power	1. Check whether the fuse in the power base is burned out. If burnt, replace the fuse. 2. Press the start button, the button will light up after it is powered.
	2. The power cord is not properly inserted or connected	2. Plug or connect the power cord properly.
The display screen does not work	1. No power	1. Turn on the power button (ON). 2. Contact your supplier.
The accessory probe does not work	1. Poor interface contact	1. Unplug the output cable, and then reinsert the socket.
Weak power for Cold & Hot hammer, scrubber, RF, Galvanic and other functions	1. The connector is not properly plugged in. 2. The accessories are too hot. Overheating can cause weak output power.	1. Adjust power to a higher level. 2. Wait for the accessories to cool down. Turn off the device when not in use.
Dermabrasion has weak suction	1. The thread of the diamond tip is not properly tightened. 2. The inside of the dermabrasion handle is obstructed. 3. The air tube is worn out. 4. The connection between the air tube and the suction handle is loose.	1. Adjust the power and the air regulation valve. 2. Replace the microdermabrasion accessories. 3. Replace the air hose with a new one, adjust the connecting piece, and change the filters.

# Troubleshooting

Fault	Reason	Solution
Sprayer with weak spraying effect	<ol style="list-style-type: none"> <li>1. The joining screw of the airbrush with the spray bottle is not properly tightened.</li> <li>2. The water receptacle is blocked.</li> <li>3. The air tube is worn out.</li> <li>4. The connection between the air tube and the suction handle is loose.</li> <li>5. There is not enough water in the tank.</li> <li>6. The hose is bent.</li> <li>7. The vacuum is not connected.</li> </ol>	<ol style="list-style-type: none"> <li>2. Adjust the spray level to a higher one.</li> <li>3. Replace the water receptacle.</li> <li>4. Replace the air hose with a new one and adjust the connecting piece.</li> <li>5. Check the water level in the bottle and add more when necessary.</li> <li>6. Connect both handles.</li> </ol>
The suction of the pen is weak	<ol style="list-style-type: none"> <li>1. The glass nozzle is not properly placed.</li> <li>2. The suction handle is obstructed.</li> <li>3. The air hose is worn out, damaged, or bent.</li> <li>4. The connecting piece between the air hose and the handle has loosened.</li> </ol>	<ol style="list-style-type: none"> <li>1. Adjust the spray level to a higher one.</li> <li>2. Replace the suction handle.</li> <li>3. Replace the air hose with a new one and adjust the connecting piece.</li> </ol>
The output of high frequency is weak	<ol style="list-style-type: none"> <li>1. The electrode is not properly placed.</li> <li>2. Wear of the electrode rod due to excessive use.</li> </ol>	<ol style="list-style-type: none"> <li>1. Adjust the power to a higher level.</li> <li>2. Replace the high-frequency accessories.</li> </ol>

Pol. Ind. La Vallan  
Camí de Garreta, 92  
08600 Berga  
Barcelona - Spain  
T. +34 93 824 90 20  
E. info@weelko.com

Our products are subject to  
continuous technical improvements.  
Weelko reserves the right to  
change or modify this manual and  
its contents without prior notice.